

# Liberty<sup>®</sup> Select

with Kinexus<sup>®</sup>

## Let's Stay Connected

With Liberty<sup>®</sup> Select 3.0 (or higher) you are connected to your home therapy team with remote therapy programming.



*Kinexus Gateway and Cyclor*

*Welcome to your Liberty Select 3.0 (or higher) cyclor, with optional Kinexus<sup>®</sup> Gateway.*

*Contact your home therapy nurse to find out if the Kinexus Gateway for remote therapy programming is supported by your clinic.*

## What is remote therapy programming?


My Settings			
My Treatment   My Settings   My Records			
therapy	patient	options	hardware
add diurnal UF	last bag option	drain exit %	
language	SI units	personal data	
USB enabled	add. drain	add. drain alert	
flow alert	flow alert sound	Kinexus Gateway	
		YES	

*The Kinexus Gateway option automatically changes to YES when the USB drive is properly inserted and the Kinexus Gateway connection is detected by your cyclor.*


*When used with a Kinexus Gateway (shown above) and USB drive ("IQdrive<sup>™</sup>"), your cyclor is capable of receiving therapy programs remotely from your clinic while you remain in the comfort of your home. Your treatment data is automatically sent to the clinic for your home therapy nurse to review.*

*See reverse for more information.*


# If I am a Kinexus Gateway user, how do I receive a remote therapy program?

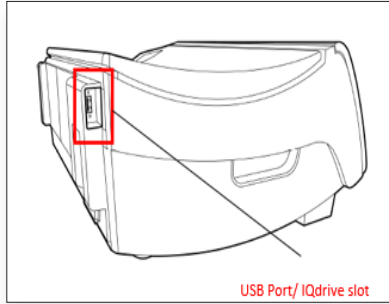


**Step 1:** Ensure the Kinexus Gateway is plugged into a wall outlet. Once powered on, connect the Kinexus Gateway to the cyclor and insert USB drive. (Review the Kinexus Gateway's User's Guide for how to connect the Gateway to the cyclor.)

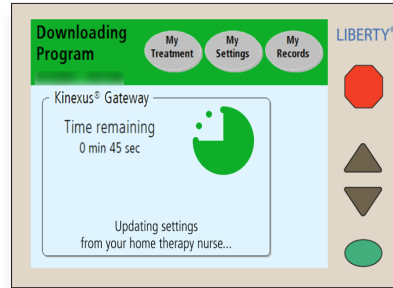

**Step 2:** Turn your cyclor on. Your cyclor will automatically begin the download if a new program is available.

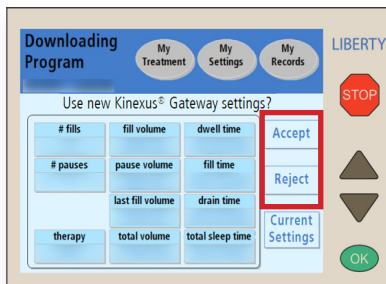

**Step 3:** When your cyclor finishes downloading the new program, you can review and accept the settings as displayed, or reject and call your home therapy nurse.

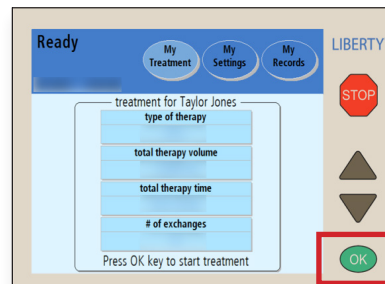

**Step 4:** After you choose Accept, your cyclor is ready to run the new program. Press OK to confirm and begin treatment.

## Support Resources Included with Your Cyclor

For more information about all of the features on your Liberty Select cyclor, and the procedure to connect your Kinexus Gateway to the cyclor (if applicable), refer to the **Procedure Card**, full **User's Guide**, and quick-reference **Handi-Guide** included in the cyclor box.

**Indications for Use:** The Fresenius Liberty Select cyclor is indicated for acute and chronic peritoneal dialysis.

**CAUTION:** Federal (US) law restricts these devices to sale by or on order of a physician.

**Use:** The Kinexus Portal is intended to allow clinicians to review patient data collected from the patient's home dialysis device(s) and to enable physicians to manage patients' home dialysis therapies for certain Fresenius products, including peritoneal dialysis cyclor(s). The Kinexus Portal does not provide medical diagnoses or recommendations regarding medical treatment. The Kinexus Portal and the content appearing on this site are not substitutes for professional medical advice, diagnosis, or treatment.

The Kinexus Gateway is intended for transmission of home dialysis therapy data and patient data for patients using certain Fresenius products, including peritoneal dialysis cyclor(s).

Kinexus is not an electronic health record (EHR) or billing solution.

**NOTE:** Read the Instructions for Use for safe and proper use of these devices. For a complete description of hazards, contraindications, side effects, and precautions, see full package labeling at [www.fmcna.com](http://www.fmcna.com).

© 2023 Fresenius Medical Care. All Rights Reserved. Fresenius Medical Care, the triangle logo, Liberty, Liberty Select, IQdrive, and Kinexus are trademarks of Fresenius Medical Care Holdings, Inc. or its affiliated companies. All other trademarks are the property of their respective owners.



RENAL TECHNOLOGIES