

# PREVENTIVE MAINTENANCE CHECKLIST SIX (6) MONTH

SERIAL NO. \_\_\_\_\_ DATE \_\_\_\_\_  
 MACHINE I.D. NO. \_\_\_\_\_ MACHINE HOURS \_\_\_\_\_  
 TECHNICIAN(S) \_\_\_\_\_ LABOR HOURS \_\_\_\_\_

See the Referenced Section of the 2008K@home™ Preventive Maintenance Procedures  
 (part number 507665 Rev. K) for detailed instructions to perform the following:

<u>Reference Section</u>	<u>Procedure</u>	<u>Value</u>	<u>Completed By:</u>
2.1	Clean Filters.....(Replace If Necessary)		_____
2.2	Inspect Pre-UF Pump Filter .....(Replace If Necessary)		_____
2.3	Inspect UF Pump Check Valves .....(Replace If Necessary)		_____
2.4	Check High Voltage AC Connections..... (<0.2 ohms)	_____	_____
2.5	Calibrate UF Pump ..... (Range: 24 ± 0.1cc at 24 UF Pump strokes)	_____	_____
2.6	Check Conductivity Calibration ..... (Range: Δ ± 0.1mS/cm)	Machine Display _____ External Meter _____	_____
2.7	Check Temperature Calibration..... (Range: 37 ± 0.3°C)	_____	_____
	(Range: 39 ± 0.3°C)	_____	_____
2.8	Check Alarm Operation and Pressure Holding Test .....		_____
2.9	Verify pH..... (Range 6.9 to 7.6)	_____	_____
2.10	Check Power Failure Alarm.....		_____
2.11	Final Checks.....		_____

NOTES: